


**DURATHANE IAB MEMBRANE
SENSATION® 7Fr., LINEAR® 7.5Fr.
and MEGA™ 8Fr. 50cc**

CARDIOVASCULAR

We created the Durathane membrane so it can stand up to your most difficult patients. It's stronger, more durable, yet thinner than any previous IAB membrane.

Durathane's patented blow-molded polyurethane material has been developed to withstand the constant abrasion of even the most calcified aortas. Yet it is so thin, it allows us to make far less invasive catheters.





- Premium Durathane membrane provides better performance*:
 - Abrasion resistance improved 43%
 - Improved fatigue resistance
 - Immediate inflation at start-up
 - Reduced insertion force

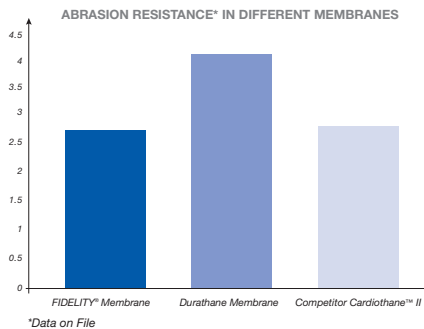

TECHNICAL TIPS FOR DURATHANE IAB MEMBRANE AVAILABLE WITH SENSATION® 7Fr., LINEAR® 7.5Fr and MEGA™ 8Fr. 50cc IABs

The membrane handles and appears different:

- Use only the introducer dilator and sheath that are packaged with the insertion kit
- Balloon prep-apply and maintain vacuum by leaving one-way valve in place until after catheter is inserted
- Keep the balloon in the T-handle until ready to insert catheter over the guidewire
- DO NOT dip, wipe, or handle membrane prior to insertion
- DO NOT twist the catheter during insertion
- Always advance in **SHORT STROKES**, continuous one inch (2.5cm) strokes (as close to the insertion site as possible) to avoid kinking the IAB catheter

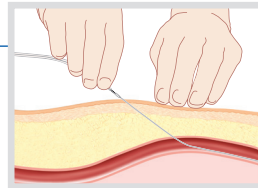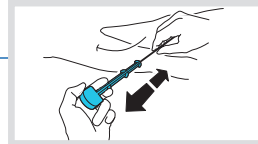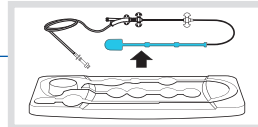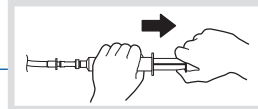

MAQUET

GETINGE GROUP

Datascope Corp.
Cardiac Assist Division
15 Law Drive
Fairfield, NJ 07004
Phone: +1 973 244 6100

For local contact:

Please visit our Website
www.datascope.com
www.maquet.com