

STATLOCK® IAB STABILIZATION DEVICE FOR MAQUET IAB CATHETERS

CARDIOVASCULAR



MAQUET, the worldwide leader in Intra-aortic Balloon (IAB) counterpulsation therapy, proudly introduces the first and only STATLOCK® IAB Stabilization Device offering needle-free securement for IAB catheters.

Custom engineered to fit MAQUET's state-of-the-art IAB catheters, STATLOCK® IAB Stabilization Device secures both proximal and distal "suture" pads without the dangers of accidental needlestick or suture wound complications.

SUMMARY OF FEATURES AND BENEFITS

- Provides custom designed STATLOCK® IAB Stabilization Device retainer for needle-free securement of MAQUET IAB catheters
- 100% elimination of suture-securement needlesticks
- Eliminates suture-wound complications (ie. bleeding, traction complaints, infection)
- Offers skin safe STATLOCK® IAB Stabilization Device adhesive system
- Designed to provide patient comfort and safety
- Allows for quick and easy application and removal

MAQUET IAB catheters and STATLOCK® IAB Stabilization Devices - where patient and clinical safety matter most.

Datascope Corp.
Cardiac Assist
15 Law Drive
Fairfield, NJ 07004
Phone: 1 800 777 4222
<http://ca.maquet.com>

Why STATLOCK?

Use **STATLOCK®** IAB Stabilization Device when you want 100% avoidance of needlesticks from IAB securement, with a quick and easy system to apply and remove.

Use **STATLOCK®** IAB Stabilization Device when your patient needs a skin-safe securement system that avoids the complications of bleeding, traction complaints, and infection.

Choose **STATLOCK®** IAB Stabilization Device for MAQUET IAB Catheters.



APPLICATION TECHNIQUE

Prep:

Prepare targeted skin area per hospital protocol. Use skin prep provided with **STATLOCK®** IAB Stabilization Device for enhanced pad adhesion and skin protection. ALLOW TO DRY (10-15 seconds)

NOTE: Always connect **STATLOCK®** IAB Stabilization Device to catheter BEFORE adhering anchor pad to skin. Arrows are oriented to point towards the insertion site.



Prep the skin



Place device on IAB catheter



Peel and secure to skin

REMOVAL TECHNIQUE

To dissolve STATLOCK® IAB Stabilization Device adhesive: Gently stroke under surface with alcohol swab while continuing to lift pad.



Disengage IAB from device



Dissolve adhesive and remove

ORDERING INFORMATION CALL 800 777 4222

Reorder #	0684-00-0472
Product Description	STATLOCK® IAB Stabilization Device for Datascope IAB catheters
Unit Quantity	1 box (20 single pouches)

For more product information, including warnings and precautions, refer to "Directions for Use" included in product packaging.

